The instruction below should be used in conjunction with detailed information on the packaging.

Name	work&care <sup>®</sup> latex gloves/
	ideall <sup>®</sup> latex
Short description of the	Latex protective gloves
product	
Full description of the	Raw material: natural rubber latex
product	Donning powder: modified corn starch (USP)
	External surface: smooth or textured
	Cuff: beaded
	Shape: ambidextrous, fitting to the right and left hand
	Sizes: S (6-7), M(7-8), L (8-9), XL (9-10)
	Colour: natural latex- creamy
	Quantity in an individual packaging: 100 pcs by weight
	Quantity in outer carton: 10 units boxes
	Expiry date: 5 years ( from manufacturing date)
Properties / Application	Latex protective gloves, classified as Personal Protective Equipment category I
	Gloves for minimal risk
	Protect against the harmful effects of cleaning agents and detergents poor
	performance and water, protect against abrasions and minor cuts
	It should take care to use gloves solely according to their intended
	application.
Contact with food	$\succ$ Gloves marked with the symbol indicating contact with food $rac{1}{2}$ , comply with
	the requirements of Regulation (EU) No 10/2011, European Regulation (EC)
	No 1935/2004 and with Regulation (EC) No 2023/2006 on good
	manufacturing practise
	Not all gloves can be used in contact with every type of food, because some
	of them can release harmful substances penetrating directly into the food.
	For more detailed information concerning intended gloves to particular
	food category, please contact the company Mercator Medical S.A.
Storage	It is recommended to store the gloves in a dry place, in the temperature of 5-
	30°C and to protect them against sunlight and fluorescent light and ozone
	Recommended relative humidity in the room where the gloves are stored is 60
	±20%
	Keep the gloves in a distance of not less than 1 m from heating devices, sources
	of fire and ozone
	Do not keep in direct vicinity of solvents, oils, fuels and lubricants
Cleaning / Sterilization	<ul> <li>Gloves are intended for disposable use.</li> </ul>
	Gloves are not intended for sterilization.
Precautions and	Before each use gloves, check and make sure that do not have visible defects
indications for use	(eg. holes, cracks, abrasions, cuts, etc.)
	Dry the hands well before putting the gloves on.
	After taking the gloves off, wash the hands thoroughly, dry them and apply a nourishing cream
	<ul> <li>Take care not to let chemical substances get under the gloves through the</li> </ul>
	cuff
	<ul> <li>If a chemical substance reaches the skin, wash it away immediately with</li> </ul>
	plenty of water with soap
	<ul> <li>If the gloves get punctured, torn or broken during their use, take them off</li> </ul>
	and put on the new ones
	and put on the new ones

	Avoid using gloves dirty in the inside as they may cause irritation an as result skin inflammation or more serious damaged.			
	<ul> <li>The gloves should not be used in contact with open fire and to protect</li> </ul>			
	against any sharp tools			
	-	ves are not intended for welding, electric shock protection, ionizing on or from the effect of hot or cold objects		
		reme caution when working with moving parts of machinery and		
		ent - there is a risk being caught in moving parts		
Components / hazardous components	-	loves may contain components known to be a possible cause of for person allergic to them, who may develop contact irritation		
components		allergic reaction.		
		made from natural rubber latex may cause allergic reactions,		
		ng anaphylactic responses.		
CE classification		ase of an allergic reaction, seek medical assistance immediately		
Product Standards	Gloves are classified as Personal Protective Equipment – category I - for minimal risk, according to Council Directive 89/686/EEC.			
Compliance				
Symbols and pictograms used on the packaging		Powdered gloves		
	(2)	Do not re-use / for single use only		
	NON STERILE	Non-sterile gloves- may cause allergic reactions, including anaphylactic responses		
		Product quality is not ensured if the packaging is damaged		
	×	Keep away from solar and fluorescent light		
	5°C	Store in temperature 5 – 30°C		
	LATEX	Latex gloves-		
	by weight	100 pcs by weight		
	LOT	Lot nr		
		Expiry date		
	M	Manufacturing date		
	***	Manufacturer: Mercator Medical S.A.		
	Rï	Product is suitable for food contact		
	P	Registered in Russia		
	Æ	Registered in Ukraine		
	21 PAP	Package made from paper, qualify for recycling		

	20 C PAP	Package made from paper, qualify for recycling		
		Be careful during opening outer carton		
	<u><u></u><u><u></u><u></u><u></u><u></u><u></u><u></u><u></u><u></u><u></u><u></u><u></u><u></u><u></u><u></u><u></u><u></u><u></u></u></u>	The direction of the carton		
	Ţ	Be careful during transport and storage		
		Package treated as municipal waste		
Utilization	substances	Used gloves can be contaminated with contagious or other hazardous substances. They should be disposed of in accordance with local regulation. Gloves should be buried or burned under controlled conditions.		
Manufacturer	Ul. H. Modi 31-327 Kral	Mercator Medical S.A. Ul. H. Modrzejewskiej 30 31-327 Kraków		
	www.merc	www.mercatormedical.eu		

Date of last revision: October 2016